


Adding Buttons and

Pages with Proloquo2Go vs. 2




1. On your iDevice, go to Settings:
 - a. Scroll on the left side of the screen to Proloquo2Go icon; touch icon;
 - b. Turn on “Show options button”

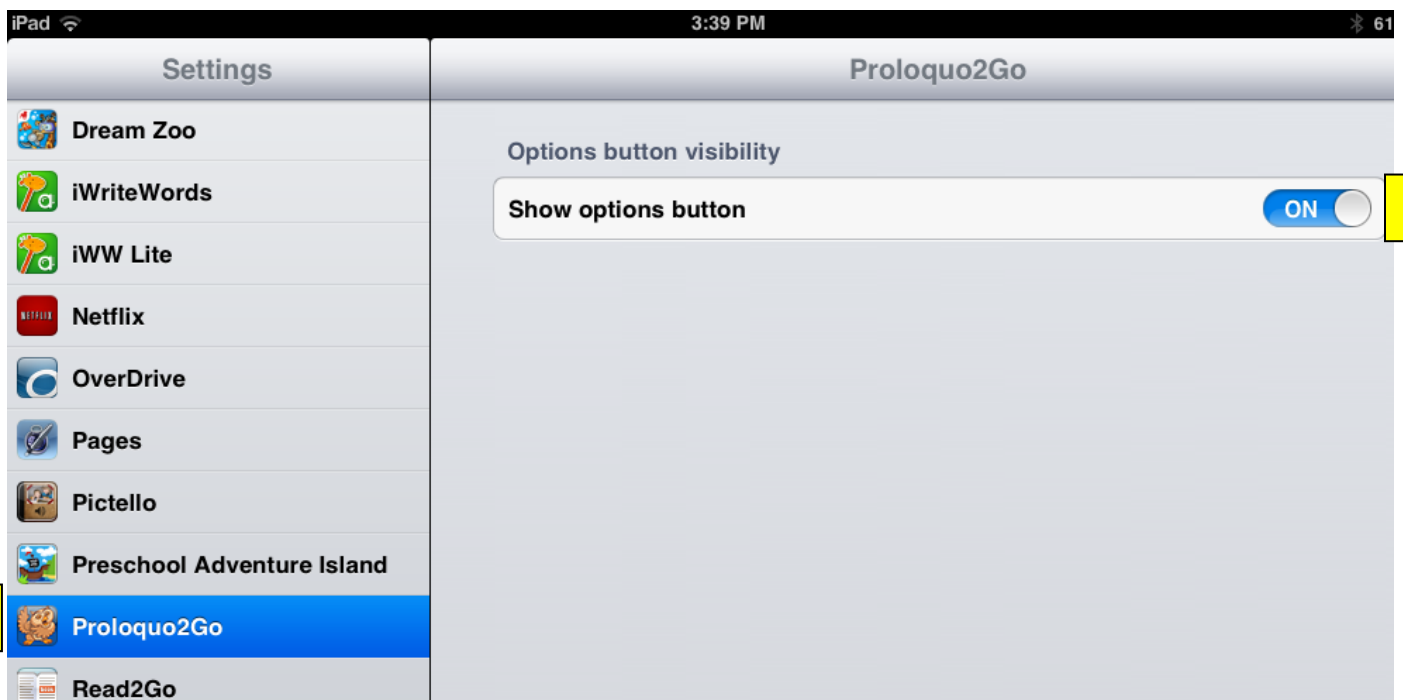


2. Press Home button to exit Settings; open Proloquo2Go

3. Touch “Options” button (gears);

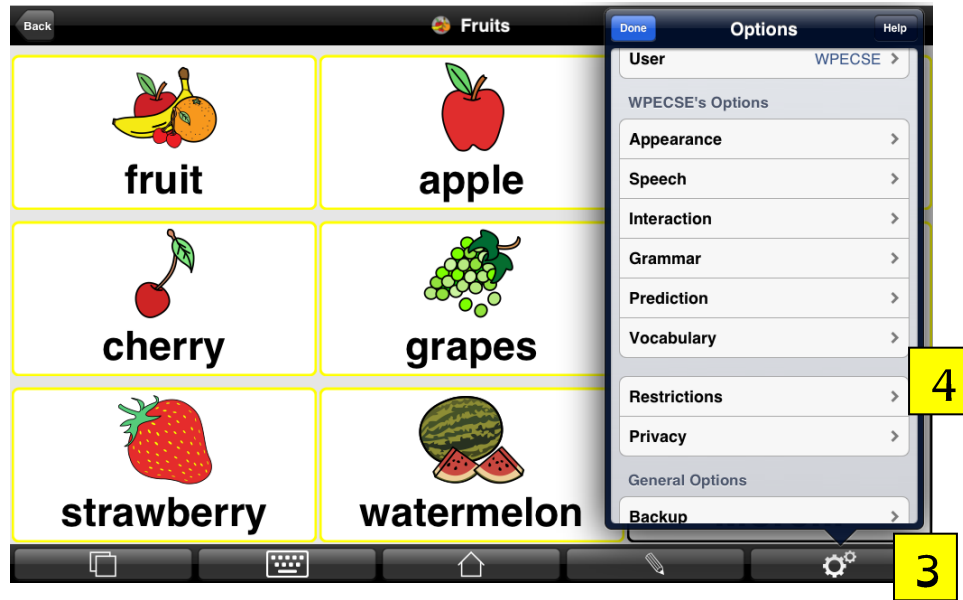


4. Touch “Restrictions”;

5. Turn on “Edit Mode” and “Adding Buttons”; touch anywhere on screen to exit “Options” mode.

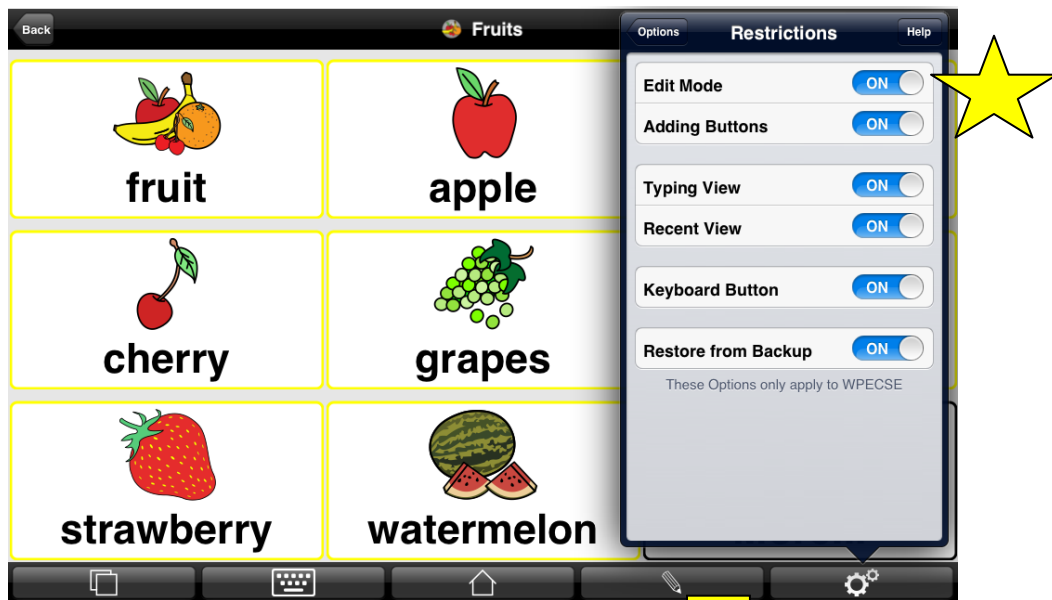


6. Touch “edit” button (pencil); editing window opens at bottom of screen.

7. To add a single button:

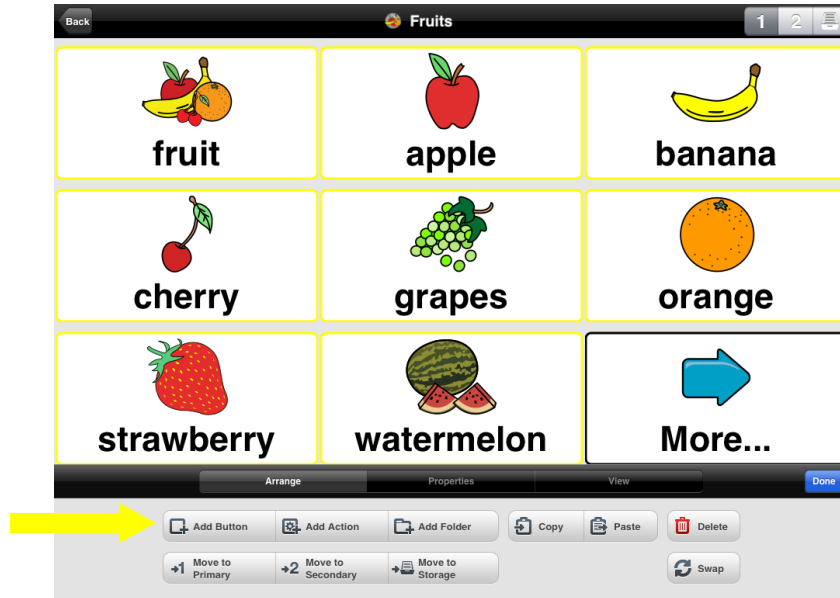


- Touch "Add button"
- Touch "Text to Speak"; keyboard appears, type what you want the button to speak

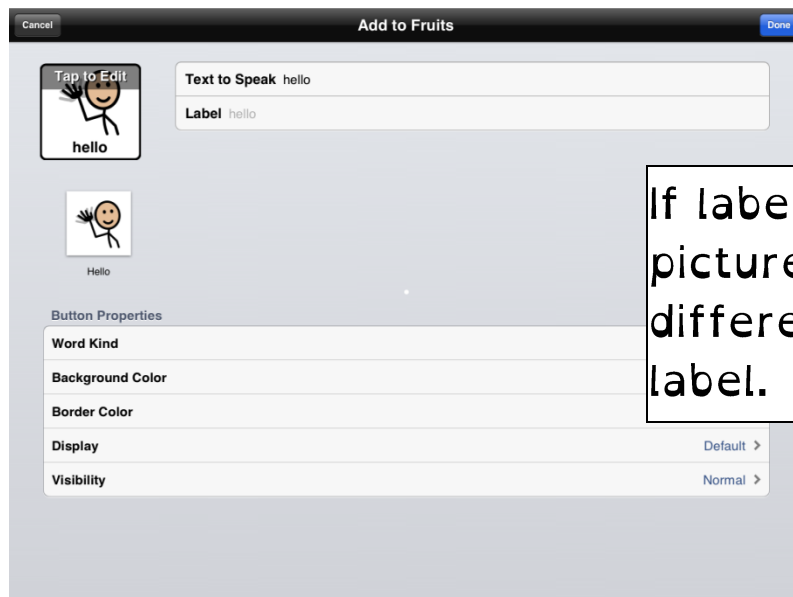


8. Press "Done" if the picture is okay.

9. To change picture, touch the picture where it says, “Tap to Edit”; a window appears with options to: Delete Picture, Choose a Symbol, Choose a Picture, or Take a Picture.

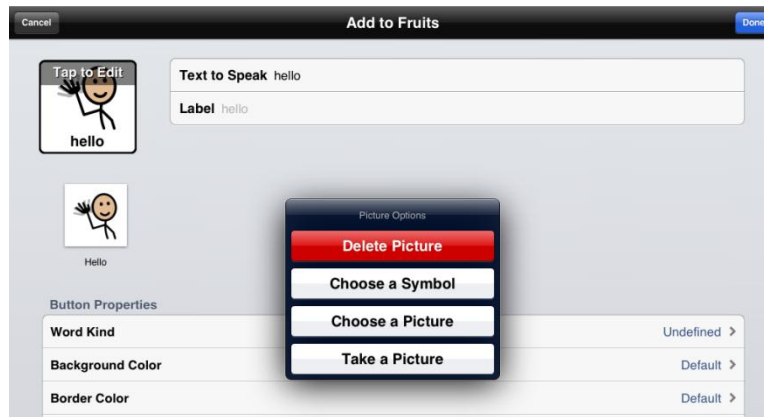

10. To add a folder (a button which will link you to a new page): press “Add Folder”; press “New Folder”; continue as with a single button.

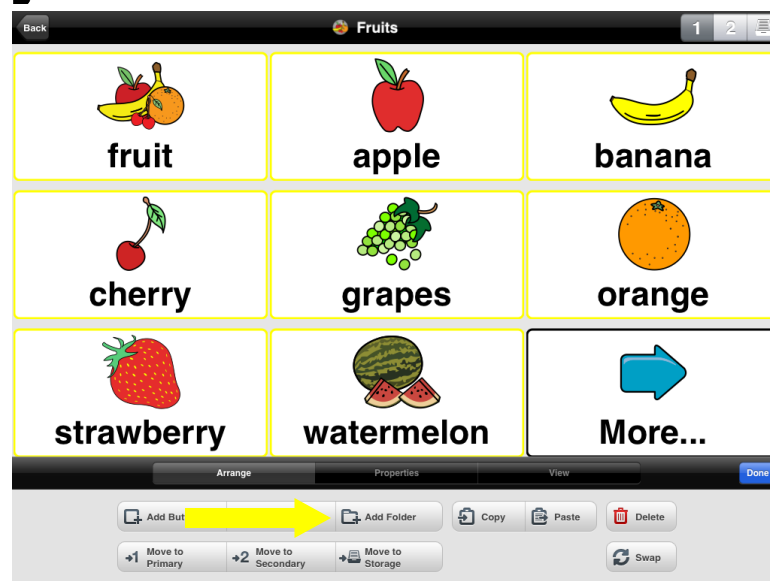

11. To change an existing button; Touch the button you want to change, and then touch “Properties” (in black menu bar); this opens a window which allows you to change the text, label, picture, visibility, etc.

